

TABS® Potty Pad Models 26600 & 26601



Features

- ▲ **Monitoring begins immediately when patient weight is sensed on the seat**
- ▲ **Maintain dignity and privacy**
- ▲ **4 ft. cord allows TABS monitor to be placed out of patient's reach**
- ▲ **Compatible with all TABS mobility monitors**
- ▲ **Available in 2 thicknesses, for compatibility with varying seat bumper heights: 1/4th inch and half inch**
- ▲ **14-day limited warranty**

NOTE: TABS Potty Pads and other personal use products are not returnable

Some things are best done alone. The TABS Potty Pad can help.

Fall risk patients or residents are safest from falls when they stay in bed or a chair. But when they enter the bathroom, they are in a relatively dangerous environment. Bathroom falls account for many serious injuries.

Using the bathroom is a very private thing, and not being able to do so alone can feel humiliating. For the caregiver, remaining in the presence of someone toileting is unpleasant as well. Fall risk patients must be escorted to the bathroom, but once seated, they can be monitored by the TABS System.

The System consists of a pressure sensitive TABS Potty Pad, placed on the underside of a toilet seat, with its cord connected to a TABS mobility monitor. The TABS Potty Pad allows the caregiver to leave the patient, and alerts the caregiver by sounding an audible alarm if the patient begins to get up unassisted. Normally, the patient will simply tell the caregiver outside the door that they are done, and the caregiver then enters to assist the patient back to bed.

To avoid contamination, TABS Potty Pads are intended for single patient use only.

