

TABS® mobility monitors are used to alert a caregiver that a patient needs assistance.

## Mounting Options

TABS Professional monitors can be easily moved from a patient bed to a chair to a wheelchair. Choose the correct bracket and mount it securely before each use.



**Wire Bracket**



**Side Chair Attachment**




**Wall Mount Bracket**



## Monitoring Options

Monitor patients with the recommended pressure pad method. Fall monitoring options include both audible alert from the monitor and/or through the nurse call system.

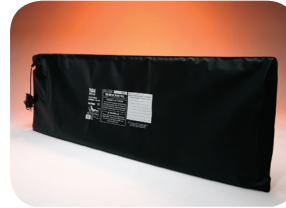

**Bed and Chair Pads**

- Record in-use date on Pad.
- Place Pad on bed or chair. For bed application, cover pad with a sheet.
- Insert phone-style plug into jack location labeled Pad on the TABS monitor.
- When the patient applies his/her weight, a beep will sound to indicate the device is alarmed.
- Alarm occurs when patient removes weight from pad.
- To silence or reset the alarm, press the black Reset button on the front on the monitor.
- Plug the nurse call cord into the jack labeled Nurse, and plug opposite end 1/4" phone jack into the nurse call outlet on the wall or bed.



## Monitor Operations

- TABS monitors are powered by a 9-volt battery located under the back cover.
- A low battery alert (intermittent beep) reminds you to replace the battery.
- To record a message:
  - 1) Remove back cover.
  - 2) Push and hold down the black record button while you speak slowly into the microphone.
  - 3) Release button when finished.
  - 4) Replace back cover.
  - 5) Review your message by pressing the reset button on the front of the alarm.

- ▲ **1069 ETL Listed**
- ▲ **JCAHO compliant**
- ▲ **Always test the TABS device and pad before each usage**
- ▲ **For full instructions, refer to TABS Professional Monitor User Manual**

**STANLEY**

**Healthcare Solutions**

www.stanleyhealthcare.com